

Usage Instructions

Assembly

Space Chamber® Standard volume: No assembly required.

Space Chamber® Large volume: Carefully insert the lug (indicated with an arrow) of the bottom half into the rectangular cut-out of the top half and push until the two halves firmly click together.

Initial use

As a static charge builds up inside the spacer, it must be 'primed' **before first ever usage.**

- Wash the Space Chamber® aerosol spacer in lukewarm water with liquid detergent. Do not rinse as the detergent helps to reduce the static charge
- Shake out excess water and allow to drip/air dry. **Do not rub dry**
- Alternatively, the Space Chamber® aerosol spacer can be 'primed' by discharging approximately 10 doses from an aerosol asthma medication ('puffer') into the chamber
- The Space Chamber® aerosol spacer may need to be 'primed' again if it is only used occasionally

On-going use

(See illustrations on right)

- Remove the cap from your aerosol asthma medication ('puffer')
- Shake the puffer several times
- Push the puffer into the base of the Space Chamber® aerosol spacer
- Attach a facemask to the mouthpiece of the Space Chamber® aerosol spacer if required
- Apply the facemask to face or seal lips firmly around the mouthpiece
- Press the aerosol canister of the puffer and then take a slow, deep breath through the mouth, hold your breath for up to 10 seconds, and then breath out slowly.
- Wait 30 seconds before repeating steps 4 – 6 for each dose from the puffer, or as prescribed by your healthcare professional. Only discharge one dose from the puffer into the Space Chamber® aerosol spacer at a time

Note

- Follow the administration directions provided with your asthma medication
- Please contact your healthcare professional if you are unsure how to clean this product
- All components of the Space Chamber® aerosol spacer are latex-free

